- Precautions:**
1. Tips are disposable; do not reuse.
 2. Do NOT allow volumes of material to be ingested. Collect extraneous material with proper vacuuming techniques.
 3. DO NOT use on patients with known sensitivity to chlorhexidine.
 4. Use Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer for a protective barrier to prevent cross-contamination.

Treatment Notes:
For composite and amalgam preparations, etc., treat surfaces the same as for temporary crown cementation. Place appropriate dentin liner and/or base as needed (we recommend Ultra-Blend® plus™).

3. Cement crown, inlay, etc., onto rinsed preparation following removal of excess water. Take seconds to bring the disinfectant closer to body temperature.

Note: If patient is not anesthetized, hold the syringe of Consepsis Scrub under hot tap water for 10

seconds to bring the disinfectant closer to body temperature. Take seconds to bring the disinfectant closer to body temperature. Note: If patient is not anesthetized, hold the syringe of Consepsis Scrub under hot tap water for 10 seconds to bring the disinfectant closer to body temperature.

Fig 5 Disinfect and clean in one directly from syringe onto tooth with a White Mac tip or into rubber cup with a cement etc., before permanent or without tip.


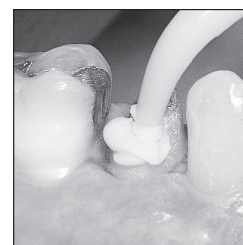
Fig 4 Express Consepsis Scrub directly from syringe onto tooth with a White Mac tip or into rubber cup with a cement etc., before permanent or without tip.

2. Deliver Consepsis Scrub directly through a White Mac tip or to prophyl cup from syringe directly to the preparation. Note Figs. 4 and 5.

Procedure:

1. Before definitive cementation, thoroughly clean preparation with Consepsis Scrub. We recommend using Ultradent Starbrushes with Consepsis Scrub for cleaning deep grooves on tooth surfaces.

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.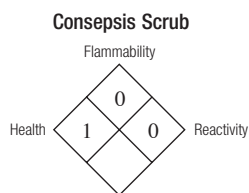

For product MSDS please visit our website:
www.ultradent.com

HAZARD RATING

- 4 = Severe
- 3 = Serious
- 2 = Moderate
- 1 = Slight
- 0 = Minimal

U.S. federal law restricts this device to sale by or on the order of a dentist.
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.
Outside U.S. call (801) 572-4200.
or visit www.ultradent.com

© Copyright 2011 Ultradent Products, Inc. All Rights Reserved.

Manufactured by:
Ultradent Products Inc.
505 West 10200 South
South Jordan, Utah 84095 USA Made in USA

23989.5 030911

ULTRADENT
PRODUCTS, INC.

ULTRADENT
PRODUCTS, INC.

**Consepsis®
Scrub**
Chlorhexidine
Antibacterial
Slurry

Description:

Consepsis® Scrub is a lightly flavored 2.0% (relative to liquid component) chlorhexidine gluconate disinfecting scrub.

Indications:

Consepsis Scrub is used for removal of residual temporary cement prior to permanent cementation and/or for cleaning debris while disinfecting.

Pre-Treatment Procedure:

A. Using 1.2ml prefilled syringes:

1. Remove luer lock cap from syringe.
2. Twist disposable tip firmly onto syringe:


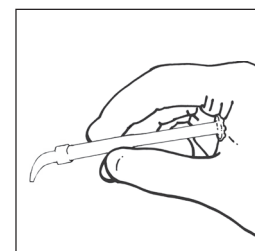
Fig. 1 Hold plunger in palm of hand for optimum control.

3. For optimum control, hold syringe in palm of hand rather than in conventional manner with thumb on plunger (see Fig. 1).

B. Using 30ml IndiSpense® System:


1. A pre-labeled unit dose syringe is attached to the end of the large, no-waste, IndiSpense syringe by turning the luer lock of the unit dose syringe snugly onto the male thread of the IndiSpense (see Fig. 2).
2. With the palm grasp (Fig. 3) the plunger of the IndiSpense syringe is depressed while the unit dose syringe is stabilized at the plunger with the nondominant hand. To prevent cross-contamination, a used syringe should not be reattached to the IndiSpense syringe. Do not express contents of unit dose syringe back into IndiSpense.
3. If not using the preloaded syringe immediately, replace luer lock cap until ready for use.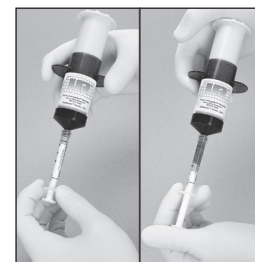


Figs. 2 & 3 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. Slightly depress plunger of IndiSpense and fill unit dose syringe to desired level.

Consepsis® Scrub Chlorhexidine Antibacterial Slurry

DE-Antibakterielles Chlorhexidin-Poliermittel/F-Pâte anti-bactérienne à base de chlorhexidine/ NL-antibacteriële chlorhexidinepasta/IT-Pasta antibatterica a base di Clorexidina/ ES-Pasta Acuosa Antibacteriana de Clorexidina/PT-Pasta Antibacteriana de Clorhexidina/ SV-Klorhexidin antibakteriell pasta/DA-Antobakteriel Chlorhexidin pasta/ FI-Antibakteerinen kloorheksidiini-tahna/EL-Αντιβακτηριακό Scrub Χλωρεξιδίνης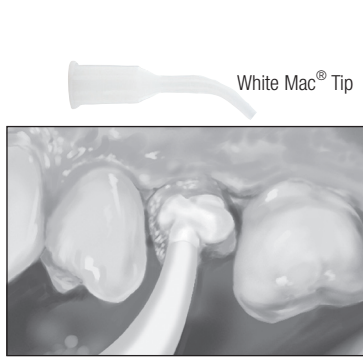

White Mac® Tip

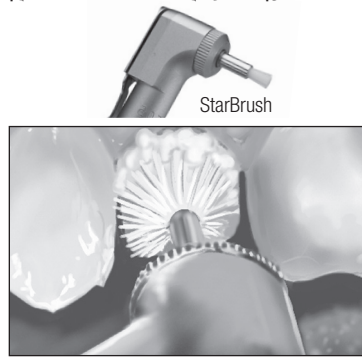StarBrush

English Deliver Consepsis Scrub directly to tooth surface.

Scour with StarBrush at slow speed to remove residual temporary cement.

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English General Precautions:

- For Professional use only.
- Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Avoid skin exposure to resins.
- Isolate strong chemicals to area of treatment.
- Confirm that patient has no known allergies to treatment materials.
- Test flow of materials from syringe and tip before using intraorally.
- Never force syringe plungers.
- Use Ultradent® Syringe Covers and/or clean and disinfect syringes between patient.
- Use standard curing lights with all light polymerized/activated materials.

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