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Use and safety recommendations

Definition

DIAStrip is a diamond coated metal strip provides two working parts in different grit sizes 40µm (red) and 15µm (white) which can be used for finishing and polishing.

Indications

Finishing and polishing of fillings and crown borders in proximal contact areas.

Recommended procedures

- The length of the metal strip (80mm) allows an optimal finger support on the adjacent teeth while working.
- To get past the contact area, use the uncoated part of the metal strip and exert a slight horizontal movement.
- Start work with the less abrasive side (white, 15 µm) in order to facilitate the access to the contact area for the more abrasive side (40 µm, red). Then begin the finishing procedure using a back-and-forward movement.
- Switch again to the less abrasive side (15 µm, white) to complete the procedure with polishing.
- The metal strip must be removed with a vertical movement after repositioning it to the uncoated part.
- If the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing the metal strip in the area to be treated.

Recommended hygiene procedures for dental practices

Protection against infection is especially important to patients and surgery staff. Disinfection and/or sterilization of used instruments are an absolutely essential part of a safe preparation technique.

- The metal strip must be sterilized prior to use. After every use it must be sterilized again.
- Keep the metal strip separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

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Coltène/Whaledent AG (Manufacturer)