Calcium Hydroxide Paste

Description:
UltraCal® XS is an aqueous, 12.5 pH, syringeable, radiopaque, calcium hydroxide paste. Use it as an endodontic treatment dressing. Superior delivery control of UltraCal XS is obtained with a NaviTip™ attached.

Indications:

UltraCal XS is to be used as per literature teachings for pulp capping, lining pulpotomies, or as a temporary dressing as per apexification procedures in endodontics.

Procedure:

Following instrumentation, remove luer lock cap from UltraCal XS syringe and attach a NaviTip. Deliver UltraCal XS short of the apex while slowly withdrawing tip (Fig. 1).

Note: Use Ultra-Blend® plus™ for pulp capping procedure.

Precautions:

1. Material is basic. Use carefully and rinse with lots of water if material is inadvertently discharged in an undesired area.
2. Tips are disposable.
3. Deliver with NaviTip, staying at least 2mm shy of the apex, always keeping the delivery tip loose in the canal. Don't deliver with plastic tips which can occlude tight in the canal (example Capillary™ tips).
4. Prefilled syringes are multi-dosage. Clean and disinfect between applications as appropriate.
5. We highly recommend using Ultradent® Syringe Covers sealed with the Impulse Sealer to provide a protective barrier against cross-contamination.

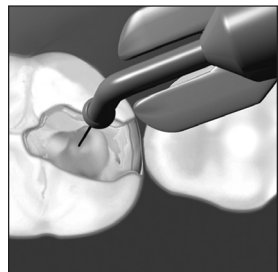
Fig. 1 Following initial instrumentation, canal is filled with UltraCal XS paste.

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Following instrumentation, fill canal from apical portion of root outward. Stay at least 2mm shy of apex.

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Canal prior to treatment on left, 5 months after application of UltraCal XS on right. UltraCal still in canals.

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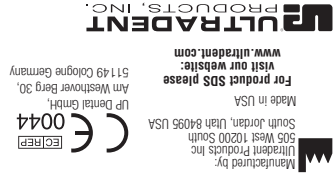
For immediate render and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200, or visit www.ultradent.com.

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one first and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

Do not re-use to avoid cross contamination. Do not repeatedly verwenden, um Kreuzkontamination zu verhindern.

FR- Ne pas réutiliser pour éviter la contamination croisée. NL- Niet opnieuw gebruiken om kruisbesmetting te vermijden. IT- Per evitare la contaminazione crociata non riutilizzare.

EN- For Professional use only. DE- Nur für professionelle Anwendung. ES- Solo para uso profesional. PT- Destina-se apenas à utilização profissional. SV- Endast för yrkesmässig bruk. DA- Kun anvendes af tandlægepersonale. NO- Skal ikke brukes av legfolk. RU- Не использовать повторно. TH- ห้ามใช้ซ้ำเพื่อหลีกเลี่ยงการปนเปื้อนข้าม

General Precautions: 1. For Professional use only. 2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed. 3. Keep products out of heat/sunlight. 4. Avoid skin exposure to resins. 5. Isolate strong chemicals to area of treatment. 6. Confirm that patient has no known allergies to treatment materials. 7. Test flow of materials from syringe and tip before using intraorally. 8. Never force syringe plungers. 9. Use Ultradent® Syringe Covers and/or clean and disinfect syringes between patients. 10. Use standard curing lights with all polymerized/activated materials.

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UltraCal XS Flammability

HAZARD RATING 4 - Severe 3 - Serious 2 - Moderate 1 - Slight 0 - Minimal

NO- Skal ikke brukes om igjen, for å unngå smitteoverføring. DE- Nicht wiederholt verwenden, um Kreuzkontamination zu verhindern. FR- Ne pas réutiliser pour éviter la contamination croisée. NL- Niet opnieuw gebruiken om kruisbesmetting te vermijden. IT- Per evitare la contaminazione crociata non riutilizzare. ES- No lo reutilice, así evitara la contaminación cruzada. PT- Não reutilizar para evitar a contaminação cruzada. SV- För undvikande av korskontaminering för produkten inte återanvändas. DA- Må ikke genbruges for at undgå krydskontaminering. FÅ- Käytä uudelleen riskikontaminointivältämiseksi.

EN- Not for injection. DE- Nicht zur Injektion. FR- Ne pas injecter. NL- Niet injecteren. IT- Non per iniezione. ES- No debe inyectarse. PT- Não injetar. SV- Ej för injektion. DA- Ikke til injektion. NO- Skal ikke brukes av legfolk. RU- Не использовать повторно. TH- ห้ามใช้ซ้ำเพื่อหลีกเลี่ยงการปนเปื้อนข้าม

General Precautions: 1. For Professional use only. 2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed. 3. Keep products out of heat/sunlight. 4. Avoid skin exposure to resins. 5. Isolate strong chemicals to area of treatment. 6. Confirm that patient has no known allergies to treatment materials. 7. Test flow of materials from syringe and tip before using intraorally. 8. Never force syringe plungers. 9. Use Ultradent® Syringe Covers and/or clean and disinfect syringes between patients. 10. Use standard curing lights with all polymerized/activated materials.

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