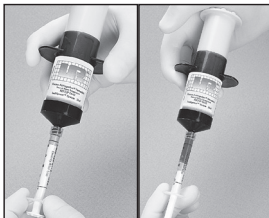
File-Eze®

EDTA Chelating & Filing Lubricant

Figs. 1&2 A 1.2ml unit dose syringe is twisted firmly onto the IndiSpense syringe. While supporting plunger of unit dose syringe with the non-dominant hand, depress plunger of IndiSpense and fill unit dose syringe to desired level.

Procedure:

1.The NaviTip® can be used to deliver File-Eze directly into the canal opening, as shown in Fig. 3. When the canal is difficult to find, start by placing point of file into canal opening prior to introducing File-Eze to pulp chamber (or onto file.) Visibility is not impaired.

2.Once canal opening is engaged, express File-Eze through tip into canal to no more than half the working length. As an alternative, a small amount of File-Eze can be placed directly onto file as seen in Fig. 4. File-Eze is only used with the first two or three instruments.

A thorough amount of sodium hypochlorite is used during entire instrumentation and cleansing procedure, alternating with liquid EDTA between each file.

4.File-Eze is irrigated and rinsed from canal, similar to other root canal preparations; ChlorCid® sodium hypochlorite solution is recommended. A slight effervescent cleansing action occurs when ChlorCid contacts residual File-Eze, which helps remove extraneous material from canal.

Precautions:

- 1.Deliver only to canal orifice. Do not express material with pressure up canal past apical foreamen.
- 2.Tips are disposed after use.
- 3.Clean and disinfect syringes between patients.
- 4.NaviTips do not have a side port hole. Do not express past apex.

- EN Filing and Chelating Agent
- DE EDTA-Gel zur Wurzelkanalaufbereitung
- FR Agent Chélatant
- NL Chelaterende gel
- IT Agente Chelante Endodontico
- ES Gel Quelante
- PT Preparação do Canal com Quelante EDTA
- SV Upplösande, Smörjande Dentin Behandlare
- DA Smøre- og chelateringsmiddel
- FI Pehmentävä ja Voiteleva Dentiiniin Käsitellyaine
- EL Παράγοντας χηλίωσης για τη ριζική
- RU Пломбирвоуный и
- PL 19% EDTA W Zelu do Chemicznego Poszerzenia Kanalu
- TR Egeleme ve şelasyon ajani
- ZH 根管治療鈣化組織處理劑

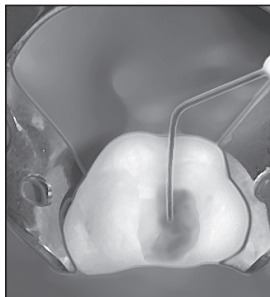
Fig. 3 File-Eze is placed after canal orifice is engaged. Visibility is not impaired.



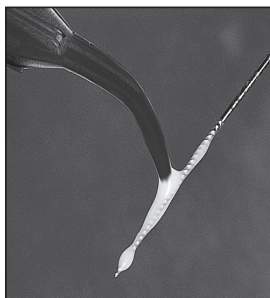
Fig. 4 File-Eze may be applied directly on file if desired.

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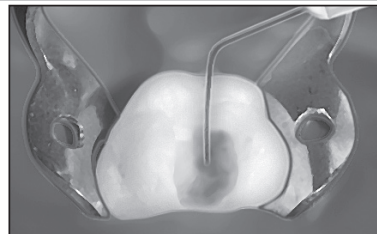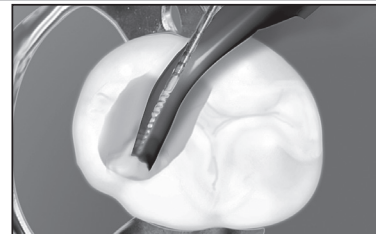Description & Indications:
Use File-Eze® as a 19% EDTA dentin chelating agent in an aqueous, water-soluble solution with a lubricating base.

Indications:
Use File-Eze to aid in instrumentation and debridement of root canals.

Procedure:
Preliminary Steps:
A. IndiSpense® system delivery:
1.The prelabeled unit dose File-Eze syringe is attached to the end of the large, no-waste IndiSpense syringe by turning the Luer Lock of the unit dose syringe snugly onto the male thread of the IndiSpense.
2.Depress plunger of IndiSpense syringe with palm of hand while stabilizing the unit dose syringe at the plunger with the non-dominant hand. Slightly pull on plunger of unit dose syringe to facilitate loading. Do not express contents of unit dose syringe back into the IndiSpense syringe. To prevent cross-contamination, a used syringe should not be reattached to the IndiSpense syringe.
3.Place cap on unit dose syringe if not using immediately, or attach a Mini tip and cover with Ultradent's TipSoc® covers.
4.When ready for use, continue with procedure steps 1-4.
B. Preloaded 1.2ml syringe delivery:
1.Remove Luer Lock cap from the syringe.
2.Twist a NaviTip® firmly onto the syringe. For optimal control, hold syringe with plunger in palm of hand, instead of in conventional manner with thumb on plunger.

© Copyright 2017 Ultradent Products, Inc. All Rights Reserved.

ULTRADENT
PRODUCTS, INC.
www.ultradent.com

For product sales please see
our website:
www.ultradent.com

Made in USA
South West, Union 94965 USA

Manufactured by
Ultradent Products Inc
3005 West 10200 South Ave
Midvale, UT 84045

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Health

Reactivity

Hazard Rating
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal

0 0 0

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a state licensed dental professional.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

EN General Precautions:
1. For professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
3. Keep products out of heat/sunlight.
4. Avoid skin exposure to resins.
5. Isolate strong chemicals to area of treatment.
6. Confirm that patient has no known allergies to treatment materials.
7. Test flow of materials from syringe and tip before using intraorally.
8. Never force syringe plungers.
9. Use Ultradent® syringe covers and/or clean and disinfect syringes between patients.

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