## Opalescence® Boost 40% Tooth Whitening System

### Description:

Opalescence Boost 40% is a 40% hydrogen peroxide, medical power bleaching gel. The jet mix syringe ensures freshness for each application and precise dosing of the activator. One barrel contains a unique chemical activation with sodium fluoride and potassium nitrate. The other barrel contains the concentrated hydrogen peroxide. When mixed, Opalescence Boost results in a 1.1% fluoride and 3% potassium nitrate concentration.

OpalDam Green is a unique, light-reflective, passively adhesive (sealing) light cured resin. Its methacrylate base is designed with sufficient strength to maintain a barrier yet is easily removed from embrasures and undercuts.

### Indications for Use:

Opalescence Boost 40% power bleaching is for medical in-office use only. It is applied by the dentist for bleaching one or more nonvital teeth, by in-office intracoronal bleaching.

Opalescence Boost is an alternative, conservative method for treating dark, internally discolored teeth (compared to crowns, veneers, etc.) caused by disease, injury, or medical treatment such as congenital, systemic, metabolic, pharmacological, traumatic, or iatrogenic factors such as dental fluorosis, jaundice, tetracycline, and adult minocycline stains, porphyria, trauma, and erythroblastosis fetalis.

Because restorative materials will not whiten, it is recommended that teeth are whitened prior to placing esthetic restorations (wait two weeks after bleaching procedures before placing adhesive restorations). Our experience shows that most treated teeth continue to whiten 12 to 24 hours after whitening treatment is completed. Bleaching problematic teeth to a more natural shade will optimize restorative shade matching.

Use Opalescence Boost alone or in conjunction with the dentist-supervised Opalescence Cosmetic Bleaching System.

OpalDam Green is used to protect soft tissue adjacent to the teeth during the in-office bleaching.

### Preliminary Procedures:

1. Clean and polish teeth approximately two weeks prior to bleaching appointment.
2. Determine and record baseline shade. Use photographs when indicated for baseline documentation.
3. Prior to mixing, bring Opalescence Boost to room temperature.
4. **WARNING:** Patient, clinician and assistant must wear protective eyewear and gloves when mixing and applying this product.

### Directions for Use:

#### Isolation - OpalDam Green (recommended isolation):

1. Remove Luer lock cap from OpalDam Green and securely attach a Micro 20ga tip.
2. Verify flow prior to applying intraorally.
3. Place self-supporting lip/cheek retractors.
4. Rinse and dry teeth thoroughly, especially the gingival third.
5. Apply OpalDam to the lingual and labial gingiva, expressing through the embrasures to assure a seal and barrier to gingiva.
6. Continue building the barrier apically until it is 4-6mm high and 1.5-2.0mm thick.
7. Express resin through any open embrasures on to the lingual tissue completely sealing and covering exposed papilla.
8. Using the mouth mirror, look up the long axis of the teeth and re-apply resin to any minute openings.
9. When protecting tissue adjacent to one or a few teeth, light cure the resin barrier on standard mode using a scanning motion for approximately 3-5 seconds per tooth, keeping the lens approx. 1 cm from the barrier and tissue. With a dental instrument, carefully check for a full cure.

#### Isolation - Rubber Dam

1. Apply rubber dam placing ligated clamps bi-laterally and one tooth distally to the teeth to be treated.
2. Before attaching rubber dam to the frame, apply a bead of rubber dam sealer or

## EN

1. Light-cured resin barrier at the gingival margin beneath the dam.
3. Attach dam to the frame and invert rubber dam at the gingival border. Check for optimum seal.

### Mixing Opalescence Boost

1. Check that syringes are securely attached before mixing. Follow pictorial diagram included in this kit.
2. Depress the red plunger forcing the contents of the red syringe into the clear syringe.
3. Quickly and forcefully, depress and seat the small clear plunger into the middle small clear syringe to rupture internal membrane and combine bleach and activator.
4. Continue depressing the clear plunger so all gel is in the red syringe.
5. Reverse the action and rapidly mix 25 times (12-13 on each side). (Fig.1)
6. Press all mixed gel into the RED syringe.
7. Twist to separate the two syringes and attach the Micro 20g FX tip onto the red syringe.

NOTE: Mixed Opalescence Boost expires after ten (10) days if refrigerated.

### Endodontically Treated Teeth:

1. Check radiograph for alveolar crest height.
2. Completely remove all filling material from the pulp chamber.
3. Place a 1-2mm glass ionomer or resin modified glass ionomer filling on pulpal floor at or coronally to the alveolar crest height. Filling material should be thick enough to completely seal the obturation orifices and material.
4. Follow "Isolation" instructions above isolating both the labial and lingual sides of the tooth. (Fig 2)
5. Follow the "Mixing" instructions above.
6. Verify flow prior to applying intraorally.
7. Express into access opening. (Fig 3)
8. Allow gel to remain in place 10 minutes.
9. Suction gel from access opening using the Luer Vac and SST (Surgical suction tip) or standard surgical evaluation tip. To avoid gel splatter, do not use water while suctioning gel. Use caution not to dislodge the isolation barrier or rubber dam seal.
10. When no gel is visible, lightly rinse and air dry. Use caution not to dislodge the isolation barrier or rubber dam seal.
11. Evaluate shade change and repeat steps 7-10.
12. If desired results are not achieved after 3 applications, discontinue treatment.
13. Follow "Clean-up" instructions to remove gingival barriers.
14. Seal access with a non-eugenol provisional restorative.
15. Wait 2 weeks before final bonding or placing esthetic restoration.

15. Wait 2 weeks before final bonding or placing esthetic restoration.
- NOTES:
- If additional whitening is desired, Opalescence Endo walking bleach is recommended.
  - Regression of whitening should be expected. Compensate by slightly over bleaching the tooth.

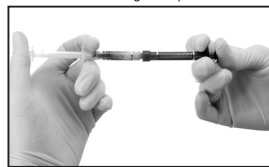
Fig. 1 Activate by pressing plunger 12-13 times each direction/side. Finish with total contents in red syringe. Remove clear syringe and dispose. Twist tip firmly onto red syringe.

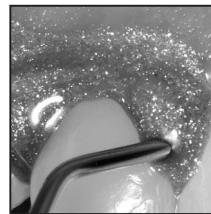
Fig. 2 Express OpalDam Green resin to build a 4-6mm by 1.5-2mm thick strip onto gingiva. Lap approximately 0.5mm onto enamel.

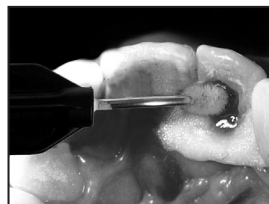
Fig. 3

## Clean-up

1. Suction gel from teeth using the Luer Vac and SST (Surgical Suction Tip) or a surgical suction tip then thoroughly rinse teeth with an air/water spray and the high volume suction.
2. Using a dental instrument, gently slide the tip beneath the OpalDam Green and lift off. Check and remove any interproximal remnants.
3. If using a rubber dam, follow Clean-up Step 1, remove clamps, dam, and frame.
4. Remove rubber dam sealer with spatula, soft toothbrush, floss, suction and firm air/water spray.

### NOTES:

- For cosmetic whitening, wait 24 - 48 hours after Boost treatment before beginning a take-home whitening system such as Opalescence PF tooth whitening gel or Opalescence Go.
- Etching teeth before any bleaching procedure is not recommended and does not potentiate the bleaching process.

### Precautions and Warnings:

1. Clinician and patient must wear protective eyewear with side shields when mixing and applying Boost
2. Carefully read and understand MSDS and all instructions before using.
3. Bring syringes to room temperature before mixing.
4. Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. If resistance continues,  
DO NOT use that syringe and contact Ultradent Customer Service for replacement.
5. Use only recommended tips.
6. Keep out of reach of children.
7. Wear protective gloves when handling this product.
8. Gingival and oral health should be confirmed before using this product.
9. Before disposing of syringes, aspirate water into the syringe to dilute and remove all residual bleaching gel. Dispose of liquid into a drain or toilet followed by 2-3 quarts/liters of water. DO NOT dispose of syringes in waste containers containing burnable materials (paper products, rubber, etc) as spontaneous combustion can result. Rinse used gauze with water.
10. Do not use on patients with a known sensitivity to peroxides, glycols, acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
11. Complete all restorative treatment prior to treatment, at least provisional.
12. If bubbles reoccur along the margin of the OpalDam Green, rinse off gel and check for leakage. Lightly dry area and re-apply OpalDam Green. If problem persists, remove dam and re-apply.
13. Cover any sensitive areas of exposed dentin.
14. Material is not to be injected.
15. Refrigerate product when not in use.
16. Keep out of direct heat and/or sunlight.
17. Mixed gel can be saved and used within 10 days. Re-connect syringes, disinfect by wiping syringe with an intermediate level disinfectant between uses.
18. Dispose of tips and syringe properly.
19. When protecting tissue adjacent to one or a few teeth, light cure the resin barrier on standard mode using a scanning motion for approximately 3-5 seconds per tooth, keeping the lens approx. 1 cm from the barrier and tissue. With a dental instruction, carefully check resin for a full cure.
20. Proper isolation techniques must be followed to protect soft tissue from the bleaching gel.
21. A quality tight seal and complete isolation on the labial and lingual cervical area of the tooth with OpalDam or a rubber dam is imperative.

**ULTRADENT**  
 PRODUCTS, INC.  
 1-800-552-5512, 801-572-4200  
 www.ultradent.com

Product MSDS see our website:  
 www.ultradent.com

Made in USA  
 South Jordan, Utah 84095 USA

Manufactured by:  
 Ultradent Products Inc.  
 505 West 10200 South

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EN-Not for injection.  
 FR-Ne pas injecter.

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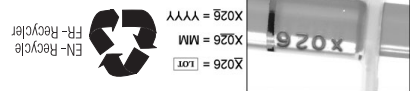
Refrigeration Required.

Imported by:  
 Clinicians Choice Dental Products  
 MNP 80031742  
 NGA 1M6

For immediate order and/or complete descriptions of Ultradent products, refer to Ultradent catalog or call 1-800-552-5512.

For professional use only. Keep out of reach of children.

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