Definition

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Definition

SwissFlex Discs: 3 of the 4 discs are made of thin, transparent foils that are selectively coated on both sides with aluminium oxide particles. The coarse disc is completely coated on the upper side with black silicon particles. The 4 grit sizes can be differentiated by the colour coding identification used on the centre of the disc, which must be placed onto the mandrel. All 4 discs are available in two diameters (9 and 13 mm). The discs are single use only. The mandrel is designed for multiple uses.

SwissFlex Strips are thin, transparent foil strips which are selectively coated with aluminium oxide particles on one side only. The black portion is coated over the entire surface with silicon carbide. The colour coding for the SwissFlex Strips is the same as that used for the SwissFlex discs.

Indications:

SwissFlex Discs and SwissFlex Strips are disposable. They are used for polishing composite, amalgam, glass ionomer cement, semi-precious metal and precious metal.

Contraindications:

Treatments during the course of which the SwissFlex Discs might jam or bend, if they are used with a rotation that is faster than 16'000 min⁻¹, and if they are used multiple times.

Recommendations for use and safe operation

SwissFlex Disc

- Use suitable DIATECH diamond finishers or the black SwissFlex Disc 703UM or 704UM (please review "Overview product range") for previous removal of excess composite and for retouching composite fillings.
- Only use SwissFlex Discs in combination with the SwissFlex Mandrel 702RA. The mandrel is suitable for multiple uses.
- For safe mounting and removal, hold instrument between middle and index finger and place on the mandrel (for removal: disengage from the mandrel), applying gentle pressure. A noticeable click confirms safe locking.
- Bring the SwissFlex Disc to the required optimal speed of 10'000 min⁻¹ prior to application to the tooth. The maximum permissible speed is 16'000 min⁻¹.
- Always provide sufficient cooling (at least 50 ml/min.); additional external cooling targeted at the operational site is required when working with the front end.
- Observe the specified sequence (black - blue – red – white)
- Avoid high contact pressure.
- Avoid contact with soft tissue.
- The operating time per SwissFlex Disc is 5 – 20 sec., according to the respective application.
- Improper use, such as excessive speed or pressure, can lead to overheating and damage to the surface to be restored as well as the destruction of the SwissFlex Disc. Risk of injury!

Overview product range

Packaging content SwissFlex Starter Kit 250095AA: per disc 25 pcs, 20 strips coarse/medium, 20 Strips fine/ultra-fine.

Content of refill packs of SwissFlex Discs and SwissFlex Strips is 80 pcs, for the mandrel 5 pcs.

	Information	contouring (coarse)	finishing (medium)	polishing (fine)	high shine (ultra-fine)
Discs	Order number	Ø 9mm: 704UM Ø 13mm: 703UM	Ø 9mm: 706UM Ø 13mm: 705UM	Ø 9mm: 708UM Ø 13mm: 707UM	Ø 9mm: 710UM Ø 13mm: 709UM
	center colour	black	blue	red	white
	grit sizes	70µm	50µm	30µm	5µm
	max. r.p.m	10'000 min ⁻¹	10'000 min ⁻¹	10'000 min ⁻¹	10'000 min ⁻¹
	active surface	upper side	double side	double side	double side
Order number mandrel		702RA			
Strips	Order number	700UM		701UM	
	colour	black	blue	red	white
	active surface	one side	one side	one side	one side
	width	3,5mm		3,5mm	
	length	175mm		175mm	
Width per grit size)	80mm		80mm		

SwissFlex Strip

- The uncoated surface in the middle of the strip serve to insert between the to be threaded area
- Prepare horizontal to the occlusal plane with constant back-and-forward movements
- Observe the specified sequence (black - blue – red – white)

Recommended hygiene procedures for dental practices

- The mandrels are to be treated with anti-corrosive disinfection and cleaning agents for rotary instruments (e.g. Biosonic Solution UC38). Regarding recommendations for use (immersion time, concentration, suitability) of the disinfecting or cleaning agents, please refer to the instructions provided by the respective manufacturers of these agents.
- Thoroughly remove all residues from disinfecting and cleaning agents using water, and allow to dry completely (e.g. by air stream). Do not leave or store mandrels in wet or damp conditions for a prolonged period of time. Inspect cleaned mandrels visually. Any damaged mandrels are to be rejected and discarded.
- Only the mandrels can be sterilized, in contrary to the disks and strips which are disposable and therefore cannot be sterilized.
- Sterilisation of the mandrels is carried out in the autoclave at 134°C. Observe instructions provided by the manufacturer.
- After completed sterilization check surfaces for corrosion. Discard any corroded mandrels.

Safety

- The discs and strips are disposable and have to be discarded directly after each use.
- The SwissFlex mandrels are suitable for multiple uses. Worn and damaged mandrels are to be discarded and immediately replaced by new ones.
- The above instructions regarding handling and cooling are to be strictly observed.

Date of issue

01-2012

Only supplied to dentists and dental laboratories or upon their instructions.

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Coltene/Whaledent AG