

VALO[®]



www.valo-led.com



VALO



VALO Cordless

VALO SUGGESTED CURING TIMES

DENTAL

POWER (mW/cm²)

POWER BUTTON


TIMING INTERVAL LIGHTS
MODE/STATUS LIGHT

TIME/MODE CHANGE
BUTTON




STANDARD

1000* ●

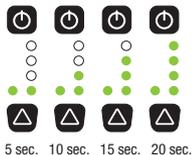

HIGH POWER

1400* ●

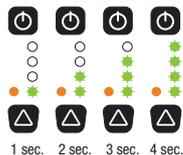

XTRA POWER

3200' ☀



PER 2mm LAYER

1 X 10 SECONDS

2 X 4 SECONDS

1 X 3 SECONDS

FINAL CURE

1 X 20 SECONDS

3 X 4 SECONDS

2 X 3 SECONDS

ORTHO

METAL & CERAMIC BRACKETS

1 X 20 SECONDS

3 X 4 SECONDS

2 X 3 SECONDS

Applicable for all current and future Ultradent and Opal Orthodontics light-cured materials.

*As measured by a properly calibrated Demitron Light Meter

†As measured by an Integrating Sphere

ULTRADENT
PRODUCTS, INC.

800.552.5512 WWW.ULTRADENT.COM

99453.2 020112

