Ease-It™

Composite Heater

- GB Instructions
- D Anleitung
- F Instructions
- NL Instructies
- I Istruzioni
- DK Brugervejledning
- E Instrucciones

INSTRUCTIONS

GB

PRODUCT DESCRIPTION

The Ease-It™ is designed for warming up dental composite materials prior to use in order to improve their handling properties.

THE EASE-IT™ PARTS:

- Heating Unit with electronic temperature control. Temperature can be set between 40 – 50°C (104 – 122°F) according to chosen material and individual preference. Default setting is at 45°C (113°F).
- Jar serving as a non-stick mixing pad. The lid with photo-bloc filter prevents premature curing of the composite material. Caution: Avoid scratching the non-stick surface. The jar can also be used for warming unit-dose tips.
- Universal thermo-block for pre-filled composite syringes. Turned upside-down, it can be used for unit-dose tips instead.

INSTRUCTIONS

Connect the cord to the Heating Unit and plug it into a 230V outlet (or 115V for item no. 20011). Place the unit-dose tips or composite syringes in the jar or the thermo-block.

When the LED indicator turns from yellow to green after approx. 5 min., the unit has reached the selected temperature.

The jar and the thermo-block are both heat-insulating and can be removed from the heating unit and placed at work site.

CAUTION: To avoid cross-contamination, do not put used (contaminated) unit-dose tips back into Ease-it™ for re-use. If the composite material is extruded into and applied from the non-stick jar, the jar must be thoroughly cleaned between patients.

CLEANING / DISINFECTION

All parts can be wiped over with a cloth with disinfectant. Do not immerse any parts into liquid. Do not autoclave.

SERVICE WARRANTY

The device is covered by a 1-year guarantee on materials and construction. In case of malfunction, please return the unit for repair over your Ease-It™ dealer.

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RØNVIG Dental Mfg. A/S · Gl. Vejlevej 59 · DK-8721 Daugaard