# CAPSU-JECT™

## Applicator

- Instructions
- Anleitung
- Instructions
- Instructies
- Istruzioni
- Brugervejledning


### INSTRUCTIONS

GB

CAPSU-JECT™ Applicator for dental materials in unit-dose capsules for direct oral application.

CAPSU-JECT™ can be used for dispensing cements, glass-ionomers etc. in manufacturer pre-loaded capsules. Please follow the instructions from the dental material manufacturer concerning activation, mixing and other special handling prior to application.

Release button  
Auslösungshebel  
Bovenste knop  
Bouton déclencheur  
Pulsante rilascio  
Udløserknap

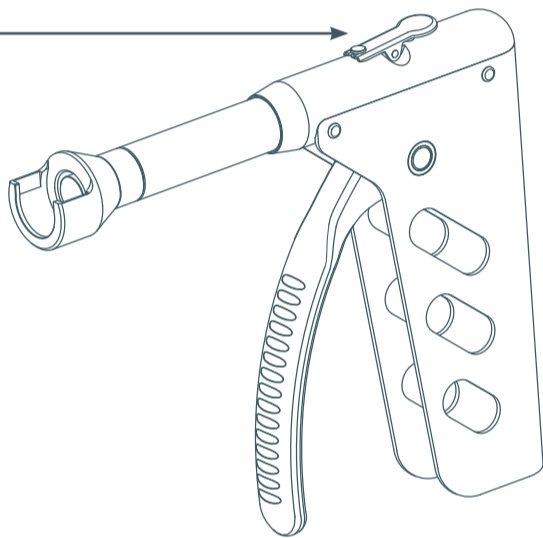1. Place a unit-dose capsule into the lock at the front part of the applicator.
2. Activate the lever to dispense the material.
3. Press the release button, by which the plunger returns to the start position.
4. Remove the capsule from the lock.

#### IMPORTANT NOTE:

CAPSU-JECT™ is designed for pre-filled capsules only. The device is not suitable for use with self-filled capsules as leakage may impair the moving parts of the device.

#### CLEANING & STERILIZATION

Clean in detergent and water and rinse with water prior to sterilisation. CAPSU-JECT™ can be sterilized in autoclave by max. 180°C/356°F.

#### MATERIALS

CAPSU-JECT™ is made from acid-resistant stainless steel.

#### GUARANTEE

CAPSU-JECT™ is covered by a 5-year material and construction guarantee. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED. B2007/GB/D-F-NL-DK-11.10.UZ.
RØNVIG Dental Mfg. A/S · Gl. Vejlevej 59 · DK-8721 Daugaard